



Saliva collection instructions


Do NOT eat, drink, smoke or chew gum for 30 minutes before giving your saliva sample.
Do NOT remove the plastic film from the funnel lid.

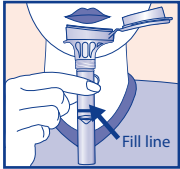


1. Spit until the amount of saliva (not bubbles) reaches the fill line.

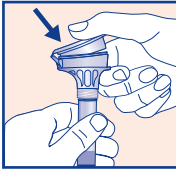

2. Close lid tightly by pushing down hard on the funnel lid until you hear a loud click.

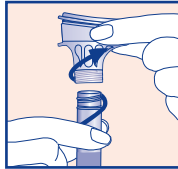

3. Hold the tube upright. Unscrew the funnel from the tube.

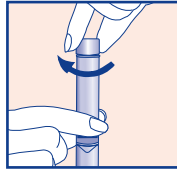


4. Use the small cap to close the tube tightly.




5. Shake the capped tube for 5 seconds. Discard funnel.

⚠ Small cap, choking hazard. Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest.

Labeling

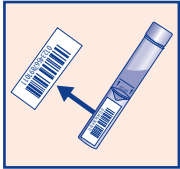


6. Locate barcode on collection tube.

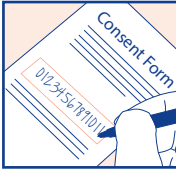


7. Write 14 digit barcode number on consent form in the space provided.



8. Write barcode on the bottom of this page in space provided for your personal record.

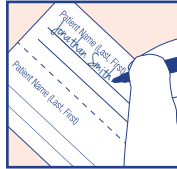

9. Write the patient name and DOB on the label provided in your kit.

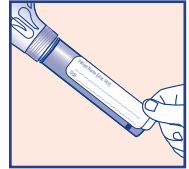


10. Apply this label to the sample tube – please do not cover the barcode.

Mailing instructions



10. Seal the tube in bag provided and place in box.

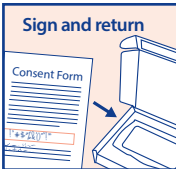

11. Fold signed consent form and place in box.



12. Close box and seal.



13. Mail from nearest USPS location (postage paid).



14. Drop in nearest FedEx drop location if return label is FedEx

Need help?

Phone:
877.395.7608
617.714.0315

Email:
support@
courtagen.com

Donor's Name and Date of Birth (MM/DD/YEAR): _____